


Treatment guide for TRIGEMINAL NEURALGIA with the Handy Pulse Laser

info@pulselaserrelief.com
1300 886 929

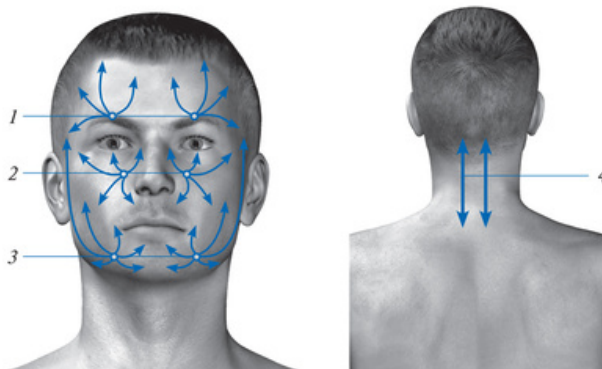

Session Procedure

No.	Exposure Area	Frequency	Exposure Time
1	Along branches of the ophthalmic nerve (starting from the supraorbital fossa and ending in the mid-sinciput), scanning	VARIABLE	1-2 min per each area
2	Along branches of the maxillary nerve (including maxillary fossa, cheekbone area, under-eye area and upper teeth alveoli projections), scanning	VARIABLE	
3	Along branches of the mandibular nerve (including the mental foramen, angle of mouth, tongue, gums, and teeth of lower jaw, skin of cheeks, lower jaw and lip and muscles of mastication), scanning.	VARIABLE	(totally, not to exceed 10 min)
4	Along the cervical department of the spine (C2-C7), paravertebral, scanning	VARIABLE	2 min on each side

Session Frequency: 1 session per day

Course: 10-15 sessions

Repeated courses: as required, in 3-4 weeks

The Short Light probe can be useful for this type of treatment